

# The BFC Complete.

(Prefilled with Vaccu•sil.)




## Instructions for use:



**1. Remove the tab.**

The tab keeps the material from mixing until ready to use.

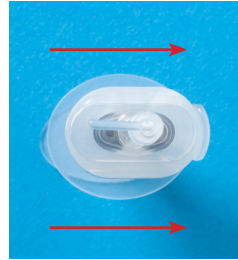

**2. Slide.**

Slide the top half all the way over until it hits the side stop and it's ready to go. Always "bleed" some material before going to the mouth.



**Better angle. Narrow tip.**

Our sharply angled tip gives you control and access to tight and hard-to-reach areas.

The narrow tip allows direct access to the entire sulcus, minimizing bubbles at the margin.



**Disposable.**

No time-consuming, messy clean-ups.



**Lower cost. Less waste, too.**

The cost is less than a mixing tip alone, yet the BFC Complete comes prefilled with Ho Dental's amazing Vaccu•sil VPS hydrophilic impression material.

Save \$2 per impression or \$1000 a year.



**Ho Dental Company**

*Order through your dealer.*