

Better. Faster. Cheaper.



Better angle. Narrow tip.

Our sharply angled tip gives you control and access to tight and hard-to-reach areas.

The narrow tip allows direct access to the entire sulcus, minimizing bubbles at the margin.



Pre-loadable and disposable.

Pre-load as many syringes as you want.

No time-consuming messy clean-ups.



Lower cost. Less waste, too.

The cost, even loaded with impression material, is less than the cost of a mixing tip.

Save \$2 per impression or \$1000 a year.

Instructions for use:



1. Initial loading position.

Our BFC syringe comes ready to load. Just place the wash material against the double-barrel and squeeze. Half-full is enough for 1 crown and full is for 3 crowns.



2. Storage position.

After loading, push the plunger in until the wash material is almost up to the O-rings. Then slide the top half until stopped by the tab.



3. Remove the tab.

The tab keeps the material from mixing until ready to use.



4. Ready to use.

Slide the top half all the way over until it hits the side stop and it's ready to go. Always "bleed" some material before going to the mouth.

